DE **EN** **FR**

Anbauset für Krankenhausnormschiene (2 Stück)
Attachment set for standard hospital rail (2 pieces)
Kit de montage pour rail normalisé d'hôpital (lot de 2)
WM 15805

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These fitting instructions describe how to fit the following brackets to a standard hospital rail:

- Wall bracket BASE-STATION 1 NG (WM 9640/9645)
- Holding plate for wall brackets (e.g. for Medumat Transport charger or Accuvac) WM 15845

Read the instructions for use for the brackets in question. They form part of these fitting instructions.

Safety information

Warning!


- Use only standard hospital rails designed for a weight of at least 7 kg, otherwise the holding plate may fall off, injuring people and damaging the device.
- If the BASE-STATION 1 NG wall bracket is used, the standard hospital rail must be adequately mounted so as to hold the bracket securely in compliance with EN 1789 pertaining to ambulances and/or RTCA DO 160E.
- Holding plate WM 15845 may only be used for in-patient applications.
- Suspending third-party products or other makes of adapter is not permitted.

Note


- The fitter is responsible for the fitting of brackets.

Fitting the attachment set to the bracket

1. Put one drop of the screw adhesive supplied on each of the screws.
2. Screw the standard rail adapter loosely to the bracket.
3. Align the standard rail adapter in the horizontal.
4. Tighten the screws.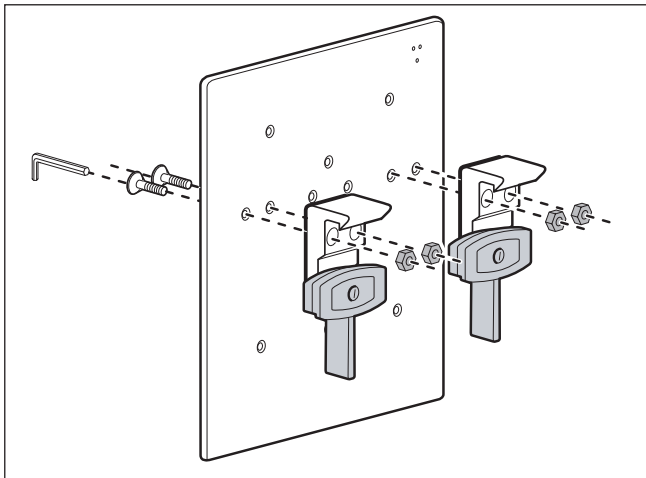

Fitting the bracket to the standard hospital rail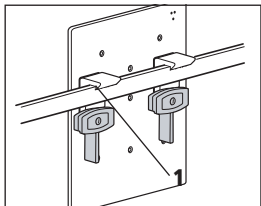

1. Suspend the bracket from the standard hospital rail using the standard rail adapters (1).


Warning!

Risk of injury if bracket is loose!


An inadequately secured bracket may fall off and injure people.

- Ensure that the latch is correctly engaged.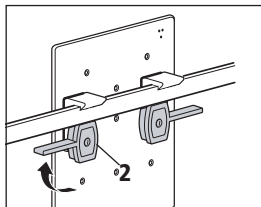

2. Push the locking levers (2) outwards until the latch obviously engages. The locking levers are now at an angle of 90° to the bracket.

Removing the bracket

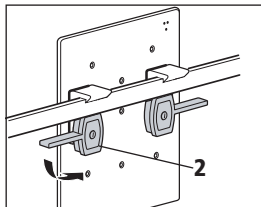1. Push the locking levers (2) of the standard rail adapters down until the latch is released.
2. Lift the bracket off the standard hospital rail.

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Geräte für Medizin GmbH+Co. KG