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These fitting instructions describe how to fit the following brackets to a rail (Ø 19 mm - 40 mm):

- Wall bracket BASE-STATION 1 NG (WM 9640/9645)
- Holding plate for wall brackets (e.g. for Medumat Transport charger or Accuvac) WM 15845

Read the instructions for use for the brackets in question. They form part of these fitting instructions.

## Safety information

---

### Warning!

- Use only rails designed for a weight of at least 7 kg, otherwise the holding plate may fall off, injuring people and damaging the device.
- The bracket with rail bracket may only be attached to round rails (Ø 19 mm - 40 mm). Any other kind of attachment, to standard hospital rails, for example, is not permitted.
- The bracket and rail bracket may only be used on wards. They may not be used in emergency vehicles.
- Suspending third-party products or other makes of adapter is not permitted.


### Note:

- The fitter is responsible for the fitting of the bracket.


## Fitting the attachment set to the wall bracket

1. Put one drop of the screw adhesive supplied on each of the screws.
2. Screw the rail bracket to the bracket.

**Note:** You can use the rail bracket for either vertical or horizontal rails. The drawing shows fitting for a horizontal rail. If you want to use a vertical rail, rotate the rail bracket through 90°.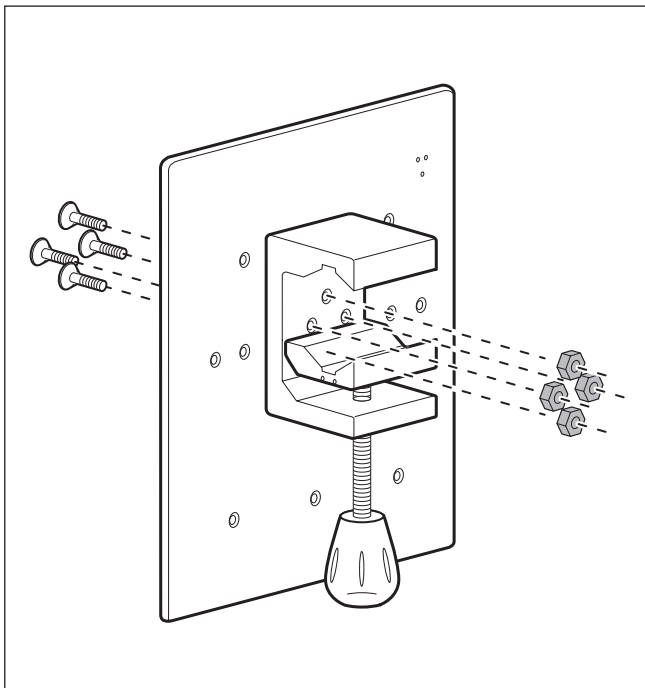

## Fitting the bracket to the rail

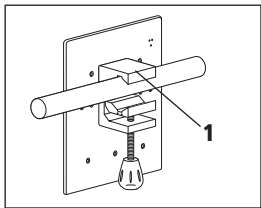1. Suspend the bracket from the rail using the rail bracket (1).


### Warning!

#### Risk of injury if bracket is loose!


An inadequately secured bracket may fall off and injure people.

- Ensure that the bracket can no longer be moved on the rail.
- Use only rails with diameters between 19 mm and 40 mm.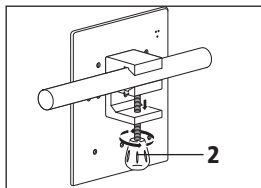

2. Tighten up the locking screw (2) of the rail bracket until the bracket can no longer be moved.

## Removing the bracket

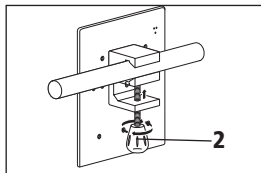1. Hold the bracket steady with one hand.
2. Slowly undo the locking screw (2) of the rail bracket.
3. Remove the bracket.

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**Geräte für Medizin GmbH+Co. KG**